

Oxford[®] Up

Safe and efficient standing,
transfers and rehabilitation



Oxford® Up

Simple and safe

The Oxford Up is a uniquely designed manual stand-aid and transfer device, suitable for those who require assistance when standing but are able to contribute effort to the process. By encouraging user participation, the Up can play a supportive role in patient rehabilitation, function, and independence.

Key Features

- Safe Working Load 200kg / 31st / 440lbs
- Disassembles for easy storage and transportation
- Adjustable legs for access around furniture
- Contoured swing-away seat pads
- Ergonomic design considerations
- Compatible sling offer for additional support

- A** Multi-point Caregiver Push Handle
- B** Patient Hand-hold Bar
- C** Attachment Point for Compatible Clip Slings
- D** Contoured Swing-away Seat Pads
- E** Soft Moulded Knee Pad
- F** Leg Adjustment Foot Pedal
- H** Braked Rear Castors (100mm)
- G** Base / Footplate with Non-slip Overlay
- I** Swivel Front Castors (75mm)



Disassembles for storage and transportation

The Up can be quickly and easily disassembled into three components to ease storage and transportation requirements. Its compact footprint when disassembled enables it to fit comfortably into the boot of most family saloons.

Reassembly is straightforward and can be completed in under one minute, without the need for tools.





Adjustable Leg Opening

The legs can be opened and closed to help navigate around bulky furniture such as commode frames and armchairs. This enables closer and safer access for the patient and better ensures good posture and positioning during use.



Contoured swing-away seat pads

Padded and shaped for comfort, each seat pad rotates outwards to allow unobstructed access for the patient when standing.

Ergonomic design considerations

The Up has been designed with the five key principles of moving and handling in mind. This direct correlation ensures Oxford products are intuitive, safe and comfortable to use.

- 1 Get close to the load
- 2 Use a wide stable base
- 3 Ensure a comfortable firm grasp
- 4 Keep spine close to neutral
- 5 Make sure movement is smooth



Soft moulded knee pad for comfort



Foot push pad helps initiate forward momentum when loaded



Contoured, multi-point push handle and patient hand-hold bar




Oxford Deluxe Standing Sling with Clips

Can be used to provide additional seated support and reassurance during transfer.



Product Dimensions

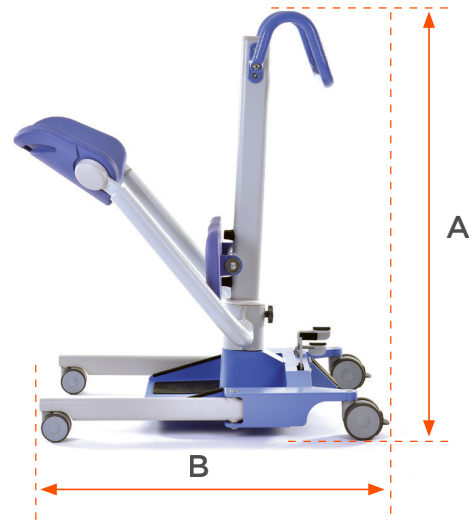

Technical Specifications

| | |
|---|----------------|
| Safe Working Load | 200 kg (31 st) |
| Overall Height (A) | 1093mm |
| Overall Length (B) | 960mm |
| Overall Width (C) | 633mm |
| Legs Open - External Width (D) | 840mm |
| Legs Open - Internal Width | 720mm |
| Legs Closed - External Width (E) | 585mm |
| Legs Closed - Internal Width | 470mm |
| Leg Height | 105mm |
| Knee Pad Height (Footplate to top of pad) | 480mm |
| Foot Plate Height | 37mm |
| Ground Clearance | 17mm |
| Turning Radius | 1100mm |
| Front Castors | 75mm |
| Rear Castors (Braked) | 100mm |
| Intended Operating Environment | +5°C to 40°C |

Weights

| | |
|--------------------|-------|
| Base | 20kg |
| Seat Support Tubes | 9.5kg |
| Mast | 4.5kg |
| Total | 34kg |

CODE: TM0600

DISPLAY NAME: Oxford Up 200

TGA NUMBER: ARTG 397217

Testing Standards

The Oxford® Up is fully compliant with the following standards and directives:

- ISO 10535
- BS EN 12182

Warranty

Please reference the user manual or contact your authorised representative for details.

Spare Parts

| NO | ENABLE CODE | DESCRIPTION | QTY |
|-----|-------------|---|-----|
| 1 | ST3970 | Mast Assembly | 1 |
| 2 | ST3971 | Knee Pad (Including Fixings) | 1 |
| 3 | ST3972 | Seat Post Assembly with Mast Inserts | 1 |
| 4 | ST3973 | Seat Pads - Left & Right Hand (Including Fixings) | 1 |
| 5 | ST3974 | Foot Pedal Assembly (Including Fixings) | 1 |
| 6 | ST3975 | Leg Assembly (Single) | 1 |
| 7 | ST3976 | Rubber Pads (Foot Tray, Pedals and Push Pad) | 1 |
| 8 | ST3977 | Plastic Moulded Parts (Including Locking Knob) | 1 |
| 9 | ST3978 | Rear Castors (Braked) (Including Fixing) (Pair) | 2 |
| 10 | ST3979 | Front Castors (Including Fixing) (Pair) | 2 |
| N/S | ST3980 | Decal Set (Not Shown) | 1 |

